

Laringospray

Always apply **Laringospray** exactly as described in this instruction or as the doctor or pharmacist has told you.


Composition: Propolis solution in ethyl alcohol, calendula flower tincture, purified water, glycerine.

Effect and use: **Laringospray** is spray developed on the base of alcohol, which is used to diminish irritation of oral cavity and pharyngeal mucosa after various manipulations have been performed or in case of inflammation. The propolis, which is the ingredient of this medical device, is one of the products of apiculture that is used in medicine most widely as an agent with anti-microbial, anti-fungal and anti-viral activity. The product covers the surface of the mucous membrane thus delaying the entry and growth of microorganisms, helping to maintain a favourable microenvironment, preventing irritation and creating the necessary conditions for the renewal of the mucous membrane so that it can better fulfil its functions.


The form of the dispenser ensures even dissemination of the product on the mucous membrane and precise dosage.

The product is developed for professional as well as personal usage.


How to use Laringospray

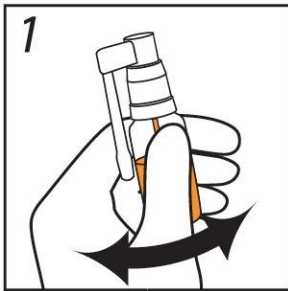


1. Shake the bottle.

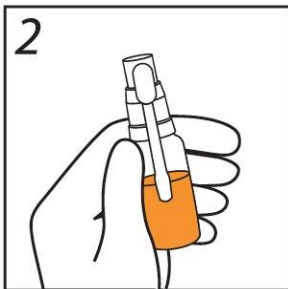

2. Take the bottle in your hand with the dispenser nozzle facing your direction.

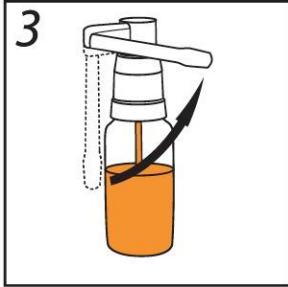

3. Turn the dispenser nozzle 90 degrees counter-clockwise.

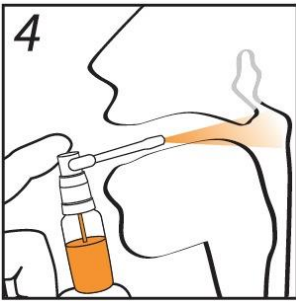

4. Place the nozzle at the mouth cavity as indicated in the figure. For the first time spray **Laringospray** in the mouth cavity by pressing the dispenser cup 2-3 times. Keep the bottle upright! Approximately 150 mg of the product are sprayed by pressing the dispenser cup one time.

5. After use, clean the dispenser nozzle with tissue paper and turn the nozzle back to the starting position.

The product is intended for use by a single user.

The product should be used 3 times a day. It is not recommended to apply the product during having a meal or while drinking to ensure prolonged covering effect.

The product is used until the symptoms of irritation and inflammation of oral cavity and pharyngeal mucosa have resolved, although if the symptoms (discomfort) persist after a week during the use of the product a doctor needs to be consulted.

Precautions for use: Do not use if you are hypersensitive (allergic) to products of apiculture or other ingredients of this product. Avoid contact with eyes! If accidental eye exposure occurs, flush the eyes immediately with flowing water and consult the doctor.

There are limited data on use in children under 12 years of age and in women during pregnancy or breastfeeding therefore the product should be applied only after consultation with the doctor.

No side effects due to the use of **Laringospray** are observed. If you get any side effects, please consult the doctor or pharmacist.

Storage: In dry place out of the sight and reach of children. To store in carefully closed original packaging, at room temperature (15-25°C). Once you start taking the product, it should be used during 3 months. Do not use after the expiry date as stated on the packaging.

Manufacturer: SIA "SILVANOLS", Kurbada 2A, Rīga, LV-1009, Latvia.
www.silvanols.com